

# JUVÉDERM® IT

JUVÉDERM® facial fillers are injectable treatments that can shape and contour many areas of the face.



Natural-looking



Lasting results, up to 24 months depending which product used



>90% patient satisfaction



The world's leading brand of hyaluronic acid facial fillers.\*

\*Based on healthcare professional tracking market research, from over 1000 healthcare professionals in the largest 13 aesthetic markets worldwide.

## Treatment areas

- Smooth lines around eyes
- Lift & project eyebrows
- Improve under-eye area
- Add volume to temples
- Add volume to cheeks
- Smooth lines around mouth
- Add volume & definition to lips
- Contour chin & jawline
- Hydrates & smooths skin



BEFORE



AFTER\*

\*Model treated with JUVÉDERM®. Images are for illustrative purposes only. Actual results may vary.

## Your JUVÉDERM® treatment today:

(Please check the relevant boxes below)

Treatment area		Products used
Eyes	<input type="checkbox"/>	<input type="checkbox"/> VOLUX™
Eyebrows	<input type="checkbox"/>	<input type="checkbox"/> VOLUMA®
Under-eyes	<input type="checkbox"/>	<input type="checkbox"/> VOLIFT®
Temples	<input type="checkbox"/>	<input type="checkbox"/> VOLBELLA®
Cheeks	<input type="checkbox"/>	<input type="checkbox"/> VOLITE™
Around mouth	<input type="checkbox"/>	<input type="checkbox"/> ULTRA
Lips	<input type="checkbox"/>	<input type="checkbox"/> ULTRA XC
Jawline	<input type="checkbox"/>	<input type="checkbox"/> ULTRA PLUS
Chin	<input type="checkbox"/>	<input type="checkbox"/> ULTRA PLUS XC
Hydrate skin	<input type="checkbox"/>	

## Post-treatment care

Your healthcare professional will review with you what to expect following treatment with JUVÉDERM®.

You may experience temporary redness, swelling, itching, pain, firmness, bumps, bruising, and/or discolouration at the injection site. If this occurs, apply an ice pack or cold compress. If the symptoms persist for more than one week, or if any other side effects develop, consult your healthcare professional as soon as possible. Please see the reverse side for other important information on possible side effects.

**You should avoid** the following for the first 12 hours:

- Applying makeup

**You should avoid** the following for 2 weeks after the treatment:

- Prolonged exposure to sun or UV rays
- Temperatures below 0°C
- Any saunas or steam baths
- Massaging the area and/or putting pressure on it for a few days

Please see reverse side for other important treatment considerations.

## Clinic contact

---

---

## Date of next visit

---

---

## Post-treatment notes

---

---

---

---

---

---

### JUVÉDERM® Safety information:

JUVÉDERM® is a family of injectable implant products intended to smooth lines and wrinkles, enhance lips, treat perioral lines and oral commissures and restore lost volume to the cheeks and jawline.

Your physician will ask about your medical history to determine if JUVÉDERM® products are appropriate for you. Tell your practitioner if you have:

- Severe allergies with a history of anaphylaxis, multiple severe allergies, allergies to hyaluronic acid or other filler products, or allergies to lidocaine or other local anesthetics.
- A history of autoimmune disease, recurrent sore throats, rheumatic fever, epilepsy, or porphyria.

The following are important treatment considerations for you to discuss with your practitioner before treatment in order to help avoid unsatisfactory results and complications:

- You are currently taking medications which prolong bleeding, such as acetylsalicylic acid (aspirin), non-steroidal anti-inflammatory drugs, or anticoagulants, as with any injection, you may experience prolonged bleeding. You should inform your practitioner about any medications you are currently taking.
- You have recently had other skin surface treatments (such as laser treatment, skin peels or dermabrasion) or have previously used fillers.
- You have skin infections or inflammation such as acne or cold sores.
- You scar frequently.
- You are pregnant or breastfeeding, or you are under the age of 18 years (under the age of 21 years if treating the lips), as the safety of JUVÉDERM® products has not been established.

Potential side effects associated with JUVÉDERM® products include temporary injection-site reactions such as redness, swelling, itching, pain, firmness, bumps, bruising, and discoloration. These reactions may occur immediately or may be delayed and may last for up to a week. Injections in the lips may cause more swelling and bruising due to the unique physiology of this area. As with any injection, there is a risk of infection. This is not a complete list of side effects. If inflammatory reactions persist for more than one week, or if any other side effects develop, seek medical attention as soon as possible.

FA-JUV-AfterCareTearPad-20E  
CA-JUV-2050019

JUVÉDERM®, VOLUMA®, VOLIFT®, VOLBELLA®, VOLITE™, VOLUX™, ULTRA PLUS, ULTRA PLUS XC, ULTRA, ULTRA XC and its designs are registered trademarks of Allergan Holdings France SAS or its affiliates, used under license by Allergan Inc.

©2020 Allergan. All rights reserved.